## SİLİKON BURUN ASPIRATÖRÜ

Silicone Nasal Aspirator

**Wee**  
baby

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EN

Wee Baby Nasal Aspirator removes nasal mucus gathered in your child's nose. The cleaning secures a calm, relaxed breathing.

**WARNING FOR THE HEALTH AND SAFETY OF YOUR BABY!**  
•ALWAYS USE THIS PRODUCT UNDER THE SUPERVISION OF AN ADULT. •KEEP ALL THE UNUSED PARTS OUT OF THE REACH OF CHILDREN.

**Instruction For Use:**

•Before each use pull to all directions and examine carefully. Do not use if damages or wearout is observed. •Before the first use boil the tip without pump for five (5) minutes. •This procedure ensures impeccable cleanliness. •Squeeze the pumping part to press out the air and move the tip of the aspirator carefully inside your child's nostril. Slowly release the pump. Nasal mucus will be sucked up and the nose will be cleaned. Repeat until all the mucus is removed. •Before each use clean the aspirator in warm and soapy water and rinse thoroughly. •After cleaning store dry in a closed container.

**WARNING:** • Not suitable for cleaning in the dishwasher. The tip without pump is suitable to be cleaned in sterilizers. •Please keep this information sheet for your reference.

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